Natural Daylight Solution
for Mobile Dental Photography

COCO Lux[®]

www.coco-lux.com

Tel: +82-2-2083-1367

Email: hasscorp@hassbio.com

HASS Corp.

77-14, Gwahakdanji-ro, Gangneung-si,
Gangwon-do, KOREA 25452
Tel: +82-70-7712-1300~4 / Fax: +82-33-644-1231
Customer Support : +82-2-2083-1367
E-mail : hasscorp@hassbio.com
Website : www.hassbio.com

Human-Aid
System Supplier **HASS** *beLIVE*

Natural Daylight Solution
for Mobile Dental Photography

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1. Introduction

Thank you for purchasing the COCOLux[®] product.

This product features natural daylight properties (color temperature 5,500K / color rendering index 90 Ra+) by utilizing LED to take advantages for accurate color determination. COCOLux[®] is designed to attach to your smartphone for photo taking.

Before use, please read this instruction manual to learn how to use the device and to ensure your safety.

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2. Safety instruction

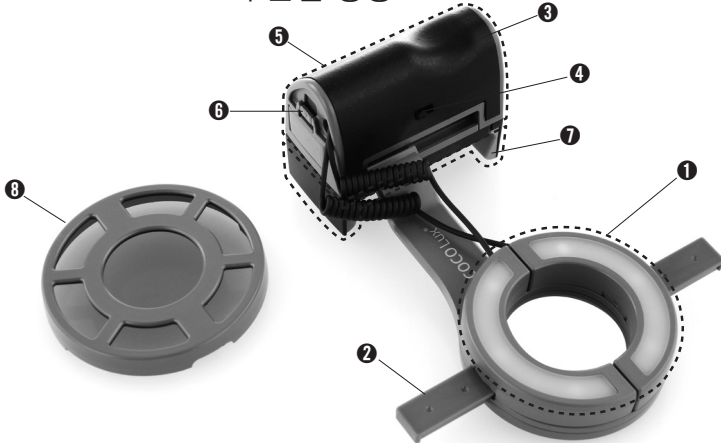
- This product has been designed exclusively for use with mobile phones. Otherwise, this may result in a malfunction or damage to the device.
- Do not use the COCOLux[®] near flammable or explosive gas.
- Use smartphones with a screen width in 55mm-89mm range.
- Refer to the example picture below when rotating the lighting unit in Twin mode. It is ONLY allowed within 90 degree range.
- Avoid the polarizing filter contacting pointed objects or being fallen to the floor. It requires extra cares to prevent any damages to it.
- Remove the 1.5V AAA batteries from COCOLux[®] if it is not in use for longer than a week.
- For more details, refer to the "IMPORTANT SAFETY INSTRUCTIONS" provided on separate paper.

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When unboxing new COCO Lux®, you will see a protection film attached to the polarizing filter. You MUST remove the film before using the filter.

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9 This pouch is to hold the product and to clean the lighting unit & polarizing filter.

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There is a pocket inside the pouch so that you are able to keep the polarizer in it when you are not using the filter.

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4. Lighting mode

Ring mode

This mode of lighting is for accurate color determination of the subject or any cases (with polarizing filter) that require high illuminance-e.g. photography of posterior teeth. It is able to take pictures with the minimized reflections when using the polarizing filter in Ring mode. It is easier to detect the difference in brightness of each tooth when the picture is converted to black and white.

Twin mode

For your dental photography work with high vividity and sharp edges, COCO Lux[®] supports you with Twin mode both horizontally (e.g. for photography of anterior teeth) and vertically (e.g. for face photography). The polarizing filter must be removed before using Twin mode. To change into Twin mode from Ring mode, shift the lighting unit along the guide bar.

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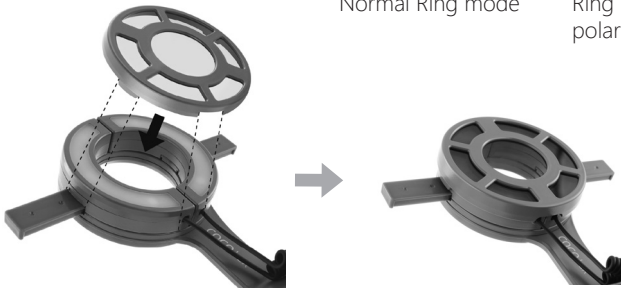
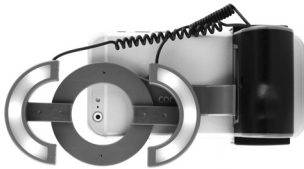
5. Recommendations for use

Before photo taking - zoom setup

Please set the camera of your smartphone with off-camera flash & 2.5-3 times zoom before your dental photography. Such zoom level will help you obtain the best quality dental pictures available on your phone.

During photo taking - how to hold the product

To prevent any instability in photo taking, please use both hands to hold COCO Lux[®] and smartphone as the example picture demonstrates.

During photo taking - the best distance

In case of Ring mode for taking shades, keep the distance around 15cm between the lighting unit of COCOLux[®] and the subject. Such distance will help you minimize unwanted reflections and obtain the optimal illuminance to take advantages in shade taking.

After photo taking - adjustment and processing of results

For the purpose of each case, you may adjust such basic properties as brightness, saturation, etc. with existing image processing mobile apps. Please visit COCOLux[®] website (www.coco-lux.com) and check the tutorials about mobile app for more details.

After photo taking-quality to share

Please set the image quality as original when sending the result via email or messengers. Otherwise, the shared images may result in compromised or damaged quality which will interfere with the efficient working process and outcome level for dental labs or other experts whom you are working with.

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